

Emdon



Mounting Instructions

Mounting

The seat is usually mounted at a height of 500mm. This is also recommended for disabled people.

Place the seat against the wall. Mark the hole pattern, pre-drill.


Screw in the screws into prepared holes.

Fold down the seat to check the clearance.

If necessary, adjust by tapping the bracket a little.

Finally, insert and fasten the two outer screws (a washer is sometimes necessary).

When mounting on an uneven wall, a washer is recommended between fittings and wall on the two outer screw holes.

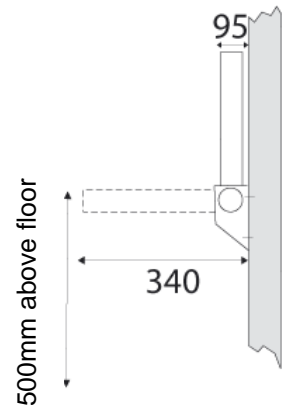

Other

The folding seat must not be mounted in a humid environment.

The fittings should be maintained with a few drops of oil on the spring.

Suggestions for different fastening methods:

Wood / Stud Wall

The seat should be mounted on load-bearing joists with a nickel-plated screw 50x5 with a round-headed countersunk head.

If necessary, use reinforcement plate.

Concrete / Brick Wall

Pre-drill for metal, wood screw or expander M5.

Lightweight Concrete Wall

The seat should be mounted with through screw and nut.

Manufacturer

Emdon AB

Giggvägen 1
296 37 ÅHUS

Tel: +46 44 249138

info@emdon.se

www.emdon.se